



The Uterine Sound is used to gauge the depth and position of the uterine cavity. It is also helpful with the placement of IUDs and the verification of the of the intrauterine position and length for HSG procedures. The proximal portion of the sound is 3.5mm in diameter tapering to 2.5 mm at the distal portion with a bulbous tip to reduce the risk of perforation. Malleable to adapt to anatomical variations. It is a sterile, disposable, single-use only device.

DIRECTION FOR USE

The Uterine Sound is a single-use sound. It is made of white, malleable, plastic material. Prior to use, the sound can be curved between the thumb and finger in order to follow the curve of the cervical canal. The graduations are in centimeters and allow for the measurement of the uterine cavity. Dispose of the Uterine Sound in accordance with all applicable Federal, State and local Medical/Hazardous waste practice

- Infection, including but not limited to Acute cervicitis and endometritis.
- Unexplained Utering bleeding.

WARNINGS

- Gently introduce and manipulate the tip of the Sound through the cervical canal. Forced entry could create a false tunnel and bleeding.

PRECAUTIONS

- Do not use if package seal is damaged or broken.
- Minor bleeding, cervical inflammation, and abdominal cramps may occur.

CONTRAINDICATIONS

This disposable Uterine Sound should not be used in the following cases:

- Possible pregnancy

CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

Do not reuse for avoiding user may be infected.