



**NOT MADE WITH NATURAL
RUBBER LATEX**

**NOTE: THE PESSARY REMOVER
IS USED AS A SUPPLEMENTARY
TOOL TO REMOVE THE PESSARY
AND DOES NOT OFFER FUNC-
TION OR TREATMENT.**

PRODUCT DESCRIPTION

The pessary remover is designed for easy removal of the pessaries. It is best used for the removal of the ring, with or without support, the oval, dish, lodge, or marland. When using the remover to remove the pessary, it must be used with lubricant.

DIRECTIONS FOR USE

1. Lubricate the introitus, and hook remover inside on the center of the pessary (will not fit through hole, however).
2. Apply gentle traction on the pessary to stabilize and bring the device toward the introitus.
3. When the pessary is within reach, compress the pessary and carefully pull it gently out of the vagina.

WARNINGS

Avoid the use of powerful force when inserting the device into the vagina. If the appearance shows discoloration, cracks or defects stop using.

RECOMMENDED CLEANING AND DISINFEC- TION/ STERILIZATION PROCEDURE.

CLEANING PROCEDURE

1. Submerge the contaminated remover with diluted detergent (rapid multi-enzyme cleaner, 1:100) for 20 minutes.
2. After cleaning, rinse, of with water thoroughly.

DISINFECTION PROCEDURE

Pour the disinfectant (CIDEX OPA) into the container to clean for 12 minutes, thoroughly rinse agent off with water.

STERILIZATION PROCEDURE

Put the cleaned remover into the autoclave to sterilize. The recommended sterilization parameters are indicated below.

- a. Pre-vacuum parameters of 132 for minutes.
- b. Gravity displacement parameters of 121C for 30 minutes or 132C – 135C for 10 minutes.